*Tender Care*

BREAST PUMP

**ASSISTANT WITH BOTTLE 180 ML**

**MANUAL INSTRUCTION**

designed in EU

V1.3

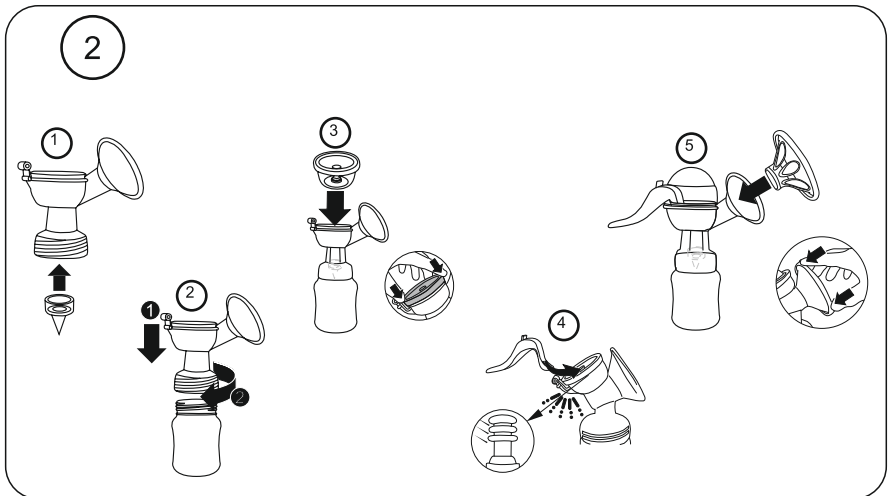
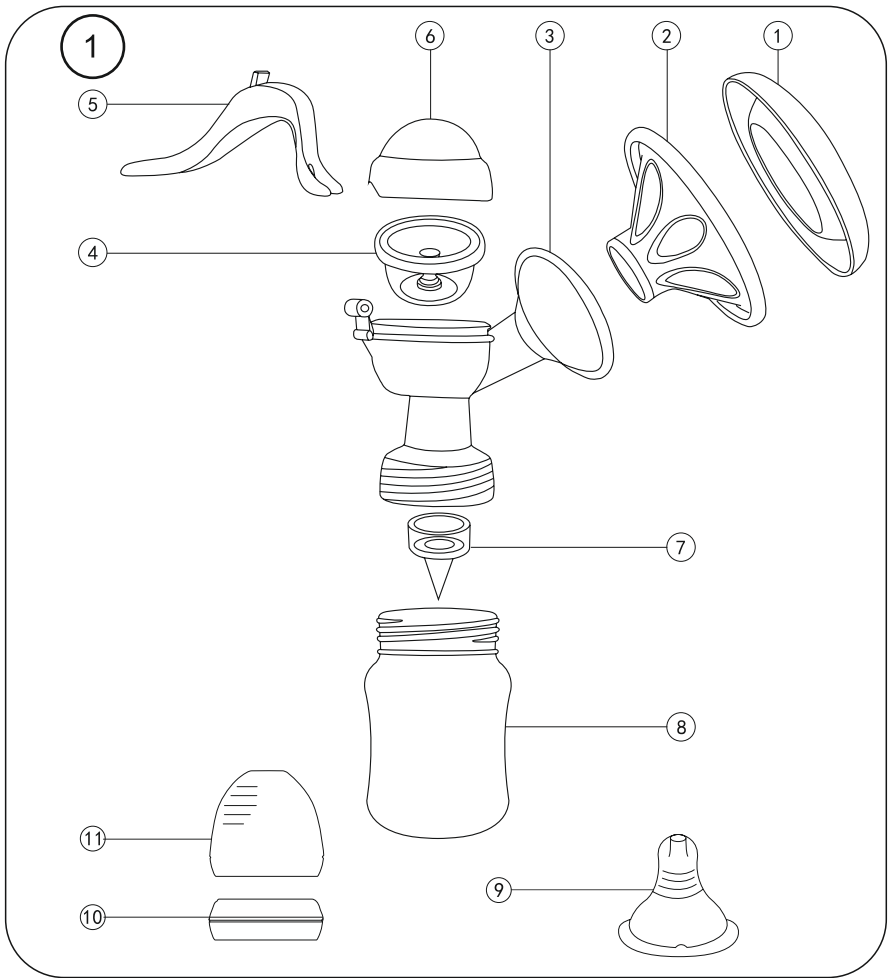
find us on

# CONTENT

<b>EN</b>	Manual Instruction.....	4
<b>BG</b>	Инструкция за употреба.....	6
<b>RO</b>	Instrucțiuni de utilizare.....	8
<b>IT</b>	Istruzione per l'uso.....	10
<b>GR</b>	ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ ΚΑΡΕΚΛΑΣ.....	12
<b>RU</b>	Инструкция по эксплуатации.....	14
<b>SRB</b>	<b>HR</b> Uputstvo za upotrebu.....	16
<b>ME</b>	<b>BIH</b>	
<b>HU</b>	Használati utasítás.....	18
<b>AL</b>	Insruksion për përdorim.....	20
<b>NL</b>	Gebruikshandleiding.....	22
<b>DE</b>	Gebrauchsanweisung.....	24
<b>FR</b>	Mode d'emploi.....	26
<b>ES</b>	Instrucciones de uso.....	28
<b>PL</b>	Instrukcja użytkownika.....	30
<b>AR</b>	دليل التعليمات.....	32
<b>TR</b>	Kullanım talimatı.....	35
<b>CZ</b>	Návod k použití.....	38
<b>MK</b>	Упатствата за употреба.....	40

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*The manual breast pump is designed for pumping and collecting breast milk from breastfeeding women. The device is designed for one user!*

**WARNING!** Never use the breast pump while you are pregnant, as pumping may cause childbirth.

#### **SAFETY TIPS**

- Do not place the pump in direct sunlight or rain, near heaters or other hot appliances, in places exposed to extremely high or low temperatures, high humidity or dust.
  - Before each use, check the breast pump for signs of damage. Do not use the breast pump if it is not working properly.
- ATTENTION:-** For hygienic reasons, the breast pump is intended for multiple use by only one user.
- Before each use, clean, rinse and disinfect all parts of the pump.
  - Only store breast milk that has been expressed with a cleaned and disinfected pump.
  - Do not attempt to remove the pump body while it is attached to the breast with a vacuum. Disconnect the vacuum connection by placing a finger between your breast and the pump funnel. Then remove the pump from the breast.
  - Never use the breast pump when you are sleepy or sleepy to avoid carelessness during use.
  - Never use accessories or parts from other manufacturers or those not specifically recommended by the manufacturer.
  - Do not pump for more than 5 minutes without interruption if no milk is leaking. Try squeezing at another time of day.
  - If the process becomes very unpleasant or painful, stop using the pump and consult your breastfeeding counselor.
  - Never drop or insert any foreign bodies into the openings.

#### **INSTRUCTIONS FOR USE**

##### **Cleaning, disinfection, storage.**

Before first use, clean and disinfect all other parts of the breast pump. Clean these parts after each subsequent use and disinfect them before each subsequent use.

- Disassemble the pump completely. Also remove the valve. Be careful when removing the valve and cleaning it. If it breaks, your breast pump will not work properly
- To clean the valve, gently rub it between your fingers in warm water with a little liquid detergent. Do not insert objects into the valve as this may damage it.
- Before first use, disassemble and clean the product and then put the components in boiling water for 5 minutes. This ensures hygiene.
- During disinfection with boiling water, do not allow the bottle or other parts to touch the side of the pot. This may cause irreparable deformation or damage to the product for which Lorelli is not responsible.
- Clean carefully before each subsequent use to ensure hygiene.
- Wash all parts with warm water and delicate detergent. Always rinse all parts thoroughly after each wash.
- Wash thoroughly and rinse all parts with clean water, including areas that may not be visible but through which the food has passed, to remove food debris. Make sure there is no liquid left in the pacifier.
- Do not place in a microwave oven. Do not heat the product in traditional ovens, on hot plates or open flames.
- Avoid the use of sharp or abrasive materials, as this may damage the product.
- Do not clean or allow the product to come into contact with solvents or aggressive chemicals. This may damage the product.
- If you use bottle brushes, be careful not to damage the most delicate parts of the product.
- After cleaning the product, dry all parts thoroughly and store in a clean and dry place without direct sunlight and away from heat sources.
- Do not leave the pacifier in direct sunlight or heat or leave it in a disinfectant ("sterilizing solution") for longer than recommended, as this may damage the pacifier. **Note:** Be sure to clean and disinfect the relevant parts of the breast pump. Wash your hands thoroughly before working on clean parts!
- Be careful, as the cleaned parts may still be hot. Only start assembling the breast pump when the cleaned parts are cooled down.

#### **PARTS**

Picture 1

1. Lid transforming into bottle pad
2. Softening pad
3. Funion
4. Diaphragm
5. Handle
6. Lid
7. Clap
8. Bottle
9. Pacifier
10. Screw on the pacifier
11. Pacifier cap

#### **ASSEMBLING THE PUMP**

Picture 2

**Important:** Before installing the pump, make sure that all parts are cleaned and sterilized.

The graphs show in detail exactly how to assemble the pump in the correct order.

Please follow them!

Wash your hands thoroughly before mounting.

1. Place the flap at the bottom of the funnel.
  2. Screw the funnel into the bottle
  3. Insert the diaphragm tightly at the top of the funnel.
  4. Mount the handle by first sliding it through the handle and then pressing it to the end until you hear a CLICK which is a sign of locking.
- CAUTION:** You can adjust the suction force by changing the position of the handle relative to the diaphragm. You have 3 positions available. Lowest position for the strongest suction, highest position for the weakest suction.
5. Apply the softening pad and lid

\* These parts are designed to fit snugly. For easier assembly, simply moisten the edge of the silicone ring and / or the softening pad with water.

## **WHEN TO SQUEEZ BREAST**

If you are breastfeeding well, it is recommended (unless your healthcare professional / breastfeeding counselor recommends otherwise) that you wait until your breastfeeding and breastfeeding schedule is established (usually at least 2 to 4 weeks after birth) before to start squeezing.

### **Exceptions:**

- If you have breastfeeding problems in the beginning, regular expressing can help to form and maintain the amount of milk.
- If you are expressing breast milk for your baby to feed in the hospital.
- If your breasts are full (painful or swollen), you can express some milk before or between feedings to relieve the pain and help the baby to latch on more easily.
- If your nipples are inflamed or cracked, you may decide to express breast milk until they heal.
- If your baby is temporarily weaned and you want to continue breastfeeding when he is with you again, it is a good idea to express regularly to stimulate breastfeeding.

## **USING THE FEED PUMP**

1. Wash your hands thoroughly and make sure your breasts are clean.
2. Relax in a comfortable chair.
3. Press the body of the assembled pump to the breast. Make sure your nipple is centered so that the massage pad can create an airtight seal. Make sure your nipple fits properly in the pillow.
4. Begin to gently push the handle down until you feel a suction from the breast. Then allow the handle to return to its original position. Note: It is not necessary to press the handle completely to the bottom, but only as far as it will go you feel comfortable. The milk will soon flow, even if you do not use all the suction power that the pump can create.
5. Repeat step 4 quickly, 5 or 6 times to stimulate the breastfeeding reflex. Warning: Do not pump for more than 5 minutes without interruption if it does not leak breast milk. Try squeezing at another time of day.
6. Create a slower rhythm by pressing the handle down and holding it down for up to 3 seconds, before returning it to its original position. Continue this way while the milk is flowing.
7. You need to pump for 10 minutes to squeeze an average of 60 - 125 ml (2 - 4 fl oz) breast milk from one breast. However, this is only an approximate amount and is different for each woman.
8. When you have finished expressing, carefully remove the breast pump from your breast and unscrew the bottle from the pump body when ready for feeding or storage. Clean and disinfect all parts of the breast pump.

## **STORAGE OF FEED**

Warning: Store only breast milk that has been expressed with a cleaned and disinfected pump to ensure hygiene.

- Breast milk can be stored in the refrigerator (not on the door) for up to 48 hours. The expressed milk should be placed in the refrigerator immediately.
- Breast milk can be stored in the freezer for up to three months if it is in disinfected bottles with a disinfected screw ring and sealing disc. Label the bottle clearly stating the date and time of expressing, and use older breast milk first.

## **WARNING!**

- Always put expressed milk in the refrigerator or freezer immediately.
- Never freeze thawed milk once.
- Never add new milk to already frozen milk.

## **BABY FEEDING**

- Inspect the feeding teat before each use and pull the feeding teat in all directions. Throw away at the first signs of damage or weakness.
- Never dip the teat in sweet substances or medication, your child may get tooth decay.
- Do not microwave. Do not heat the product in traditional ovens or on hot plates or open flames.

## **For your child's safety and health**

### **WARNING!**

1. Continuous and prolonged sucking of fluids will cause tooth decay.
2. Tooth decay in young children can occur even when non-sweetened fluids are used. This can occur if the baby is allowed to use the bottle for long periods through the day and particularly through the night, when saliva flow is reduced or if it is used as a soother.
3. Always check food temperature before feeding.
4. Throw away at the first signs of damage or weakness.
5. Keep components not in use out of the reach of children.
6. Never attach to cords, ribbons, laces or loose parts of clothing. The child can be strangled.
7. Never use feeding teats as a soother.
8. Always use this product with adult supervision.
9. Accidents have occurred when babies have been left alone with drinking equipment due to the baby falling or if the product has disassembled.

## **TROUBLESHOOTING**

**Problem:** I feel pain when I use the breast pump.

**Solution:** The possible cause is excessive suction. You do not need the full force of the vacuum produced by the pump. If the pain does not subside, stop using the breast pump and consult your breastfeeding counselor. If the pump does not come off the breast easily, you can break the vacuum connection by placing a finger between the breast and the massage pad.

**Problem:** I don't feel any suction.

**Solution:** Check that you have assembled the breast pump correctly. Also make sure that the pump is placed correctly to the breast to create a vacuum.

**Problem:** The breast pump makes a squeaking sound.

**Solution:** Make sure that you have assembled the breast pump correctly and that the handle is wet when assembling the breast pump.

**Problem:** The breast pump is scratched.

**Solution:** With regular use, it is normal for light scratches to appear and they do not cause problems. However, if any part of the breast pump is severely scratched or cracked, stop using it and contact us.

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MOTHER&BABY PRODUCTS CO.,LTD.
PRODUCTS CO.,LTD. Zhejiang, China
Importator:Didis Ltd.