Baby Care

Art.No.1022078 - S

Art.No.1022079 - L

<b>EN</b>	Silicone breast shields size S/L	1
<b>BG</b>	Силиконови протектори за кърмене размер S/L	2
<b>GR</b>	Προστατευτικά σιλικόνης για θηλασμό - μέγεθος S/L	3
<b>SRB HR ME BiH</b>	Silikonski štitnici za dojenje - veličina S/L	4
<b>RO</b>	Protectii de silicon pentru alăptare - mărime S/L	5
<b>ESP</b>	Protectores de silicona para lactancia - talla S/L	6
<b>RU</b>	Силиконовые накладки для грудного вскармливания – размер S/L	7
<b>IT</b>	Paracapezzoli in silicone per allattamento - taglia S/L	8
<b>AL</b>	Mbrojtës silikon për ushqyerjen me gji - Madhësia S/L	9
<b>HU</b>	Szilikon védők szoptatáshoz - S/L méret	10
<b>NL</b>	Siliconen tepelhoedjes – maat S/L	11
<b>FR</b>	Protecteurs de mamelon en silicone - taille S/L	12
<b>DE</b>	Silikonprotektoren für Stillen – Größe S/L	13

find us on

[www.lorelli.eu](http://www.lorelli.eu)

## Silicone breast shields size S/L

- They are made of fine, soft silicone that is BPA, odor and taste free.
- Protects sore, sensitive or painful nipples while breastfeeding
- Follows the natural shape, feel and movement of the mother's breast, allowing baby to latch and feed easily
- The shape fits perfectly and allows your baby more contact with the breast
- Easy to clean, can be sterilised
- Size S/L

**Set included:**

1/2 pcs Silicone shields

2/Storage case

3/Product dimensions: 8,5x5,5x2,5 cm ; 7gr(2x3.5gr)

4/Nipple diameter: S - 15mm / L - 20mm

**Warning!** Before using this product, please read carefully the manual instructions included inside and retain them for future reference! This product is not a toy! Always use the product under the supervision of an adult! Keep protectors out of the reach of children when not in use. Check thoroughly before each use. Discard at first sign of damage or wear.

**Please read carefully the manual instructions and retain them for future reference!**

**Warning!** This product is not a toy! Always use the product under the supervision of an adult! Keep protectors out of the reach of children when not in use. Do not leave in direct sunlight or heat, and in disinfectant for longer than recommended, as this may damage the product. Check thoroughly before each use. Discard at first sign of damage or wear. The nipple shield is a product for personal use and to prevent cross-contamination, must not be used by more than one mother.

**Cleaning and sterilization:** Before the first use, put in boiling water for 5 minutes. This is done for hygiene purposes. Leave to cool. Clean before each use. Always wash the product immediately after use. Clean all parts of the product. Wash in warm soapy water and rinse with clean water, or wash in a dishwasher (top rack only). Dry thoroughly. Do not clean with strong chemicals or solvents. It can be sterilized by steam sterilization (in an electric or microwave sterilizer), boiling or cold sterilization. After sterilizing, leave the product to cool.

**Instructions for use and storage:** Wash your hands thoroughly with soap and water. Clean the breast and surrounding areas. Make sure that the protector fits comfortably and does not move before offering the breast to the baby. Make sure that the shaped part of the nipple shield matches the position of your baby's nose while feeding. After eating, remove the protector.

Wash with warm soapy water after each use. Rinse with clean water. Dry with a clean cloth. Storage the product in its storage case in clean, dry place away from sources of heat and light.

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