www.lorelli.eu 

Baby Care™
NASAL ASPIRATOR
LORELLI

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WARNING! BEFORE USING THIS PRODUCT, PLEASE READ CAREFULLY THE MANUAL INSTRUCTIONS INCLUDED INSIDE AND REMAIN THEM FOR FUTURE REFERENCE!

Contents

Introduction	3
Explanation of Symbols	4
Important Information Before Use	5
Product Identification	7
Batteries Installation	8
Instruction for Use	9
Cleaning and Disinfecting	12
Troubleshooting	14
Technical Specifications	15
EMC Tables	16

Introduction

- 1. Music Function:** 12 melodies in the memory of this device may interest and distract a baby and help to make the clearing procedure more comfortable.
- 2. High Power:** Put aspiration tip into nostril, then you may use this device to remove nasal secretions continuously or intermittently.
- 3. Safe:** The soft aspiration tip is for comfortable design to prevent the possibility for injuring when using this operate.
- 4. Simple & Compact Design:** This device is compact easy to hold and operate. Its is also safe and comfortable for baby use.
- 5. Washable suction head:** Tips, collection cup & head part can be taken off from the device and washed.
- 6. Noiseless:** Easy to use while the baby is sleeping.
- 7. Battery:** This device needs to use two 1.5V AA batteries; it is convenient to change the batteries anytime, any where.

Explanation of Symbols

	Remark / WARNING
	Symbol for "ENVIRONMENTAL PROTECTION" - Disused electrical appliances should not be disposed of with household waste, please pass them where there are recycling facilities, please contact your local authorities or retailer for recycling advice.
	Power button
	Type of protection of the applied part of electric shock
	Temperature limits
	Manufacture's name and address
SN	Serial number of the product
LOT	Lot number
	Keep at dry place
IP 22	Protection class
EC REP	European Authorized Representative
CE	CE Mark with Registration Number. This indicates the conformity of Directive 93/42 / EEC for medical devices
RoHS	This product fulfilling the RoHS Directive 2011/65/EU

Important Information Before Use

The device is designed for using intermittent suction to remove nasal secretion and mucus in Children (age 0~12 years old) at home environment.

- 1. WARNING – Do not modify this equipment without authorization of the manufacturer.**
2. Do not expose the device to extreme temperature, very high humidity level, or direct sunlight.
3. Avoid extreme stress, shock or accidental drop-down.
4. Please replace the all batteries when the power or/and the sound of the pump getting weak during using the device.
5. It is recommended to remove the batteries if the device will not be used for a period of time.
6. The application of pressure by the device is intermittent. You shall keep pressing the power button to start the device to generate vacuum for removing nasal secretion and mucus. If you want to stop the suction, just need to release the power button to turn off the device.
7. The Nasal Aspirator is for cleaning/removing the nasal secretion only. Do not use this device other than the intended purposes. Any further medical action should follow the advice of doctors.
8. During the use of the device, the baby should not hold his breath, close his mouth or swallow his saliva to avoid vacuum pressure cause eardrum unbalanced.
9. If the baby does not feel well when using the device, stop and consult a doctor immediately.
10. Do not apply excessive force when placing the aspiration tip into the nasal cavity.
11. This device is for home use only. Please consult a doctor in case of nasal diseases. Clear the aspirator tip prior to each use.



Important Information Before Use

- 12.** The Intended Operators for this device shall be parents or adults who can follow the instructions and notices in user's manual to operate the device. The device is not allowed to be used by children themselves.
Keep the nasal aspirator away from the children. Do not let them operate the device themselves.
- 13.** DO NOT use the Nasal Aspirator when patients have the following contraindications:
- Nasal bleeding or any open wounds inside
 - Acute head, facial, or neck injury
 - Coagulopathy or bleeding disorder
- 14.** Before using the device, please read this User's Manual carefully. Wrong use of the device may cause the following hazards/complications:
- Mechanical trauma
 - Nasal irritation/bleeding
 - Mucosal hemorrhage
 - Discomfort and pain
- 15.** Please keep the replaceable tip at where children and pets cannot access to avoid risk of them been swallowed.
- 16.** The tip may cause potential allergic reactions.

CONTENT OF THE PACKAGING

Description	Pcs.
• Aspirator	1
• Instruction	1
• Tip	2
• Bateries 1.5V AA	2


Product Identification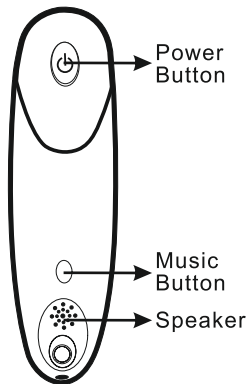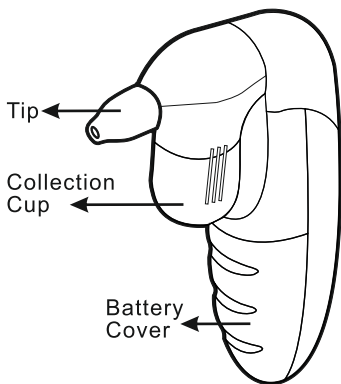

Batteries Installation

Replacing the Battery:

1. Gently slide down the battery cover back.
2. Carefully remove the old batteries and properly discard.
3. Insert or replace 2 x 1.5 V AA alkaline batteries into the battery compartment, ensuring to match the indicated polarity symbols.
4. Always use new batteries.
5. Replace the battery cover .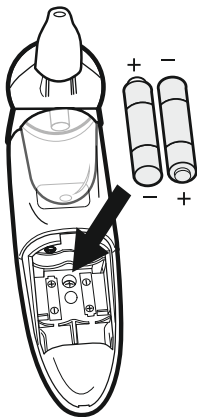

WARNING

Battery-operated

1. Please properly dispose of the batteries away from small children and heat.
2. It is recommended to remove the batteries if the unit will not be used for an extended period of time.

Batteries must be disposed of in accordance with local environmental and institutional policies.


Dispose of used batteries in accordance with the applicable legal regulations.

Never dispose of batteries in the normal household waste. (2002 / 96 / EC)

Instruction for Use

1. Choose the correct tip to use, according to shape and size of the nostrils of your baby.
 - The large tip is recommended for use when the mucus is in a more solid form.
 - The narrow tip is recommended for use when the mucus is loose and in a liquid form.


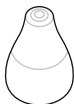
large tip


narrow tip

2. Be sure to place the tip securely on the aspirator.
Make sure tip is completely over the rib before using.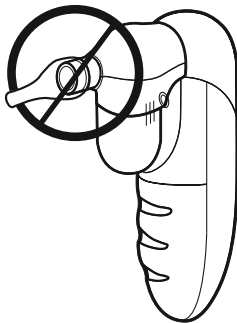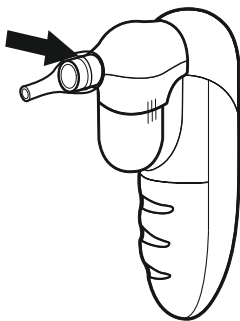

Instruction for Use

3. Press and hold the power button to allow the suction to clear the mucus.

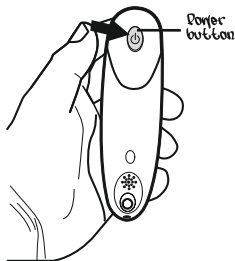
4. Hold the baby upright and insert the tip into one nostril of the baby. Be careful not to place the tip too far into the nostril (about 10mm depth) or press against the inside wall of the nostril. Be sure to keep the tip of the aspirator at a 90° angle. Do not point it up into the nostril. Allow the suction to clear the mucus. Repeat in other nostril and as needed.

WARNING

The mucus may not immediately appear in the collection cup. After each use it is recommended to give a quick clean of the tip and collection cup by removing them from the unit.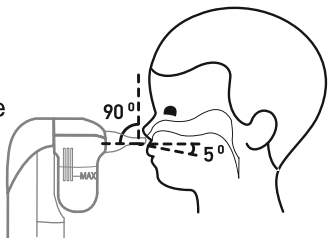

Instruction for Use


5. Release Power Button:

Release Power button to stop suction.

6. If you want to play the songs to comfort your baby, just push the music button to start . Next pressing will change to next song.

Music: Instead of the doll/noising sound when using the device, our device provides the unique design of the chord ring and 12 worldwide children's ballads.

With the beautiful songs, the baby will feel more comfortable treated with the device.

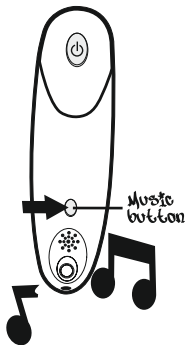
7. After using the device, press the "Music" button for 1~2 seconds to turn off the melody.


WARNING! Make sure that children could not use the appliance without supervision, because some small parts can get into the respiratory tract or the digestive system.

Cleaning and Disinfecting

1. Separate the tip and the head module from the base of the aspirator by gently pulling it apart.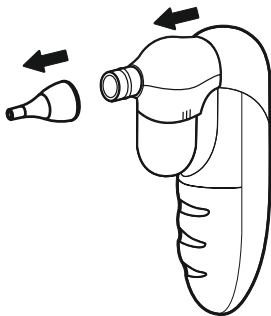


2. Unscrew collection cup by turning it clockwise.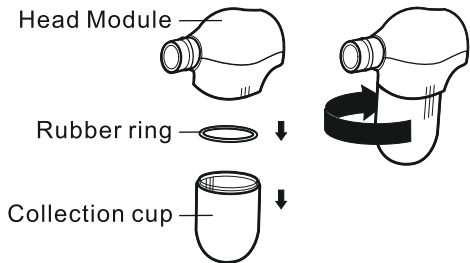

Cleaning and Disinfecting

Cleaning process of Collection cup and Aspiration tip:

Washing the collection cup and aspiration tip with mild soap in water for 3 minutes. Then rinse these parts with running hot tap water for 3 minutes and air dry.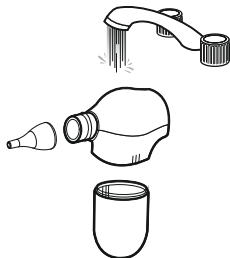


The head module can only be washed by water, it may be rinsed with hot water but not be cleaned with any soap or chemical cleaners. Soap and cleaners may damage the vent which against the mucas be sucked into the device. It is recommended to allow all pieces to dry completely before reassembly.

⚠ It is forbidden to use detergents!

WARNING! Ensure that all components are completely dry before reassembling the appliance.

⚠ DO NOT wash this device with water or any kind of liquids. It is strongly recommended to wipe the device with soft cloth and keep the device clean.

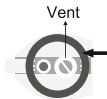Troubleshooting

Problem	Possible Cause	Solution
The suction is weak or there is no suction at all	Batteries are low or improperly installed.	Check or replace batteries.
	The rubber ring located where the collection cup screws in is missing.	Make sure there is a rubber ring where the collection cup screws in.
	The collection cup is not properly screwed in.	Make sure the three guides on each side of the collection cup line up with the guides on the suction portion of the aspirator.

WARNING

- If the device does not work properly even after you have made the above procedures, contact the merchant, from which you purchased it or with your local distributor.
- Never use the device if some of its parts are damaged, do not work properly or missing.

Technical Specifications

Effective Vacuum Pressure:

≥300mmHg(39.99kPa)

Sound Pressure Level: ≤75 dB

Motor: 3V/DC

Operating environment :

16°C~35°C (60.8°F~95°F)

with relative humidity up to 85% (non condensing).

Storage/Transportation environment :

-25°C~+55°C (-13°F~+131°F)

with relative humidity up to 85% (non condensing).

Atmospheric pressure: 700 ~ 1060 hPa

Aspiration Tip: Soft Silicone

Battery : Two 1.5V alkaline AA Size

Dimensions : 93.5 x 39.9 x 148mm (LxWxH)

Weight: 132g (without batteries)

Intended Use(s): Remove nasal secretion and mucus in Children (age 0~12 years old) at home environment.

Music: 12 Melody. The product is intended for continuous operation.

* **Note:** The manufacturer reserves the right to change the design of the device

EMC Tables

Guidance and manufacturer's declaration- electromagnetic emissions

<p>The Nasal Aspirator Lorelli is intended for use in the electromagnetic environment specified below. The customer or the user of the Nasal Aspirator Lorelli should assure that it is used in such an environment.</p>		
Emissions test	Compliance	Electromagnetic environment-guidance
RF emissions CISPR 11	Group 1	The Nasal Aspirator Lorelli uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The Nasal Aspirator Lorelli is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Not applicable	

EMC Tables

Guidance and manufacturer's declaration- electromagnetic immunity

The **Nasal Aspirator Lorelli** is intended for use in the electromagnetic environment specified below. The customer or the user of the **Nasal Aspirator Lorelli** should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment-guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV air	± 6 kV contact ± 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

EMC Tables

Guidance and manufacturer's declaration-
electromagnetic immunity

The **Nasal Aspirator Lorelli** is intended for use in the electromagnetic environment specified below. The customer or the user of the **Nasal Aspirator Lorelli** should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment-guidance
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2,5 GHz	3 V/m	<p>Portable and mobile RF communications equipment should be used no closer to any part of the Nasal Aspirator Lorelli, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance</p> $d = 1.2 \sqrt{P}$ $d = 1.2 \sqrt{P} \quad 80 \text{ MHz to } 800 \text{ MHz}$ $d = 2.3 \sqrt{P} \quad 800 \text{ MHz to } 2,5 \text{ GHz}$ <p>where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,^a should be less than the compliance level in each frequency range.^b</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol: </p>

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

- a. Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the **Nasal Aspirator Lorelli** is used exceeds the applicable RF compliance level above, the **Nasal Aspirator Lorelli** should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the **Nasal Aspirator Lorelli**.
- b. Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

EMC Tables

Recommended separation distances between portable and mobile RF communications equipment and the Nasal Aspirator Lorelli

The **Nasal Aspirator Lorelli** is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the **Nasal Aspirator Lorelli** can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the **Nasal Aspirator Lorelli** as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d=1.2\sqrt{P}$	80 MHz to 800 MHz $d=1.2\sqrt{P}$	800 MHz to 2.5 GHz $d=2.3\sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations.

Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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EC Certificate
Directive 93/42/EEC Annex II, excluding Section 4
Full Quality Assurance System
Medical Devices

Registration No.: HD 60099060 0001

Report No.: 10049126 001

Manufacturer: AVITA Corporation
9F, No.78, Sec.1, Kwang-Fu Rd.
San-Chung District
24158 New Taipei City
Taiwan

Products: Medical Products
(see attachment for products and additional site included)
Replaces Approval, Registration No.: HD 60075906 0001

Expiry Date: 2020-03-02

The Notified Body hereby declares that the requirements of Annex II, excluding section 4 of the directive 93/42/EEC have been met for the listed products. The above named manufacturer has established and applies a quality assurance system, which is subject to periodic surveillance, defined by Annex II, section 5 of the aforementioned directive. For placing on the market of class III devices covered by this certificate an EC design-examination certificate according to Annex II, section 4 is required.

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TÜV Rheinland LGA Products GmbH is a Notified Body according to Directive 93/42/EEC concerning medical devices with the identification number 0197.

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**Attachment to
Certificate**

Registration No.: HD 60099060 0001
Report No.: 10049126 001

Manufacturer: AVITA Corporation
9F, No.78, Sec.1, Kwang-Fu Rd.
San-Chung District
24158 New Taipei City
Taiwan

Products:

- Infrared (Ear, Forehead, Temple) Thermometers
- Medical Digital Thermometers
- Blood Pressure Monitors
- Nasal Aspirators

Site included:

AVITA (Wujiang) Co., Ltd.
No. 858, Jiao Tong Road, Wujiang Economic Development Zone,
Jiangsu Province, P.R. China

Manufacture of a.m. Medical Devices

Date: 2015-03-12

Declaration of RoHS Compliance

Company Name : AVITA Corporation, 9F, No.78, Sec. 1, Kwang-Fu Road, San-Chung District, New Taipei City 24158, Taiwan, R.O.C.

Product Item No : NS1

Description : Nasal Aspirator

For our model NS1, delivered to DIDIS Ltd as 1022054 NASAL ASPIARATOR LORELLI:

1. We hereby declare that the above model is compliant to the Restriction of Hazardous Substances in Electrical and Electronic Equipment Directive (RoHS) 2011/65/EU.
2. We hereby agree that only RoHS compliant materials and components will be used in the assembly of the above model. Under no circumstances will non-compliant materials and components be used in place of approved compliant equivalents during the life of the product.
3. Should it be necessary, we agree to supply documentation demonstrating compliance to the above directive on individual components or materials used in the assembly of the above model as and when requested by the DIDIS LTD BULGARIA. This will be in the form of a test report from an independent accredited test laboratory.

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Hereby we, AViTA Corporation, 9F, No.78, Sec. 1, Kwang-Fu Rd., San-Chung District, 24158 New Taipei City, Taiwan, R.O.C, declare that our product with Item number NS1, delivered to DIDIS Ltd as 1022054 NASAL ASPIRATOR LORELLI complies with the requirements of COUNCIL DIRECTIVE 93/42/EEC of 14 June 1993 concerning medical devices. Enclosed herewith the relevant certificate issued from laboratory TUEV Rheinland.

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Фирма : AVITA Corporation, 9F, No.78, Sec. 1, Kwang-Fu Road, San-Chung District, New Taipei City 24158, Taiwan, R.O.C.

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Manufacturer

AVITA(WUJIANG)CO.,LTD
Address:No.858,Jiao Tong Road,
Wujiang Economic Development Zone,
Jiangsu Province, P.R.C.

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MIB Technology Ltd.,
JPA Brenson Lawlor House ,
Argyle Square,
Morehampton Road,
Dublin 4 , Ireland

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Didis LTD, Bulgaria